

ALCOHOL WIPES FOR RAPID SURFACE DISINFECTION

Incidin™ Alcohol Wipe

READY-TO-USE WIPES FOR EFFICIENT DISINFECTION OF MEDICAL DEVICES

The absence of active residues combined with broad efficacy spectrum in short contact times make Incidin Alcohol Wipe ideal for use in healthcare settings, delivering disinfection results and compliance levels mandatory to restrict the spread of pathogens in the healthcare environment.

Incidin™ Alcohol Wipe provides:

- Broad spectrum efficacy, including Norovirus, combined with short contact times.
- Efficacy claims confirmed using extracted wipe liquid (ie. the efficacy is ensured for the liquid released from the wipe).
- High quality, dimpled wipe material for efficient application.
- Excellent material compatibility on alcohol resistant surfaces.
- Dye and fragrance free.



Incidin™ Alcohol Wipe

Ready-to-use wipes for efficient disinfection of medical devices

BENEFITS OF INCIDIN ALCOHOL WIPES

Broad efficacy spectrum inc. Norovirus efficacy in short contact times:

Broad efficacy enabling use in daily routine and outbreaks, combined with short contact times thus ensuring rapid disinfection. The efficacy on surfaces is confirmed for the liquid released from the wipe.

User friendly: Convenient alcohol wipe for rapid surface disinfection, no preparation required, available in customised pack formats.

Wide material compatibility: Excellent material compatibility on alcohol-resistant surfaces relevant for healthcare settings.

EFFICACY PROFILE

INCIDIN™ EFFICACY READY-TO-USE ALCOHOL WIPE	CONTACT TIME
Disinfection of surfaces acc. to VAH 2015 (includes test acc. to EN 16615)	
with mechanical action (bactericidal, yeasticidal, tuberculocidal) clean & dirty conditions	5 min
EN norms (clean & dirty conditions)	
EN 16615 (bactericidal, yeasticidal)	1 min
EN 13727 (bactericidal)	1 min
EN 13624 (yeasticidal)	1 min
EN 14348 + EN 16615 (method) (tuberculocidal)	1 min
EN 14476 (enveloped viruses)	30 sec
EN 14476 (limited spectrum virucidal)	5 min
EN 14476 Adeno / Noro	5 min
EN 14476 Polyoma / SV 40	10 min
EN 14476 Rota	30 sec
Efficacy according to DVV / RKI guideline	
Effective against enveloped viruses (incl. HIV, HBV, HCV) according to RKI recommendation 01/2004*	30 sec
Limited spectrum virucidal plus*	10 min
Adeno*	10 min
Noro*	10 min
Rota*	30 sec

* DVV 2015 (suspension test)

ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
Incidin™ Alcohol Wipes	6 x 100 wipes flowpack	XXXXXX
Incidin™ Alcohol Wipes	6 x 50 wipes XL flowpack	XXXXXX
Incidin™ Alcohol Wipes	6 x 90 wipes canister	XXXXXX
Incidin™ Alcohol Wipes	12 x 90 wipes refill	XXXXXX

APPLICATION AREA



Incidin™ Alcohol Wipe is a fast acting, ready-to-use wipe for disinfection of alcohol-resistant medical devices in healthcare settings.

INSTRUCTIONS FOR USE

Take single wipe. Wet surface thoroughly, ensuring contact with entire surface. Let it dry and do not rinse. Close lid after use.

The use of a disinfection wipe does not replace regular cleaning.

ACTIVE INGREDIENT

100g of liquid contains
35g 2-Propanol, 25g 1-Propanol

STORAGE

Product should be stored in a cool, dry location below 25°C and outside of sunlight. Avoid temperature extremes. Shelf life after opening is 2 months.

USE MEDICAL DEVICES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE.

FOR PROFESSIONAL USE ONLY.

ECOLAB EUROPE

Richtstrasse 7
8304 Wallisellen
Switzerland



ECOLAB DEUTSCHLAND GmbH

Ecolab-Allee 1
40789 Monheim am Rhein
Germany