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- DIRECTIONS FOR USE**
1. Squeeze equal lengths of Base and Accelerator pastes onto the mixing pad.
  2. Mix thoroughly for 20-30 seconds.
  3. Insert paste into (temporary) crown or bridge and seat onto the prepared dry tooth.
  4. Allow to set (3 minutes from start of mixing) and trim excess.
- Note:
1. Setting time can be adjusted by using more or less Base (More Base-faster set ; less Base-slower set). Do not go outside Base / Accelerator ratio range 3 : 2 to 2 : 3).
  2. If material sticks to tooth or metal, Vaseline may be added up to 25% of total mix.

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CE 0086

GC

MANUFACTURED by  
GC DENTAL PRODUCTS CORP.  
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

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Prior to use, carefully read the instructions for use.

EN

FREEGENOL

TEMPORARY PACK

EUGENOL FREE TEMPORARY CEMENT

For use only by a dental professional in the recommended indications.

**RECOMMENDED INDICATIONS**  
Cementation of temporary crowns and bridges.

**CONTRAINDICATIONS**  
In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

**DIRECTIONS FOR USE**  
1. Squeeze equal lengths of Base and Accelerator pastes onto the mixing pad.  
2. Mix thoroughly for 20-30 seconds.  
3. Insert paste into (temporary) crown or bridge and seat onto the prepared dry tooth.  
4. Allow to set (3 minutes from start of mixing) and trim excess.  
NOTE:  
1. Setting time can be adjusted by using more or less Base (More Base-faster set ; less Base-slower set). Do not go outside Base / Accelerator ratio range 3 : 2 to 2 : 3).  
2. If material sticks to tooth or metal, Vaseline may be added up to 25% of total mix.

**STORAGE**  
Store at temperature of 4-25°C (39.2-77.0°F).  
(Shelf life: 3 years from date of manufacture)

**PACKAGES**  
1-1 pkg : 55g Base, 20g Accelerator, Cleaner and mixing pad.

**CAUTION**  
In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised : 09/2008

MANUFACTURED by  
GC DENTAL PRODUCTS CORP.  
2-285 Toriimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32. 16. 39. 80. 50.

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