

SYSTEM

zetaplus



A MATTER OF TRUST

Clinical

C-Silicone Impression Material

Zhermack[®]

CLINICAL



zetaplus system

Solutions for all clinical situations.

ZHERMACK reaffirms the success of the **ZETAPLUS SYSTEM**.

The **ZETAPLUS SYSTEM**, has been appreciated worldwide for 25 years, has maintained its high performance and constant reliability in order to obtain the best possible clinical results.





REDUCED MARGIN OF ERROR

› *Clear colour contrast* to *reduce the risk of error*, consequently saving time and material.

BALANCED REACTIVITY

- › *Ideal balance* between working time and setting time. (*Snap-set* effect).
- › Optimal dosage of the catalyst increases the available working time and reduces contraction.

“FLOW CONDITIONING” EFFECT

› Increased *flowability and extreme dimensional stability* are achieved thanks to SAS (Synthetic Amorphous Silicas) high-activity spheroid fillers.

zetaplus system

“Flow conditioning” effect for high-d

ZETAPLUS with ORANWASH L and THIXOFLEX M

ZETAPLUS, soft to mix with rigid set

The *“flow conditioning” effect* helps the material *flow* and *reduces pressure* on the soft tissue. Due to its rigid set, **ZETAPLUS** is the recommended *putty* for the two-stage technique.

ORANWASH L, hydrophilic and fluid

This light bodied hydrophilic flowable silicone is suitable for the two-stage technique.

THIXOFLEX M, thixotropic, flowing and hydrophilic

This *medium bodied thixotropic silicone guarantees accurate detail* in humid environments thanks to its *intense hydrophilic character*. *Recommended for the single-stage technique*.



Soft to mix due to the “flow conditioning” effect



Flowability in the sulcus

Detail impressions.

ZETAPLUS SOFT with ORANWASH VL

ZETAPLUS SOFT, flowability and elasticity


This *soft to mix*, with *elastic setting putty* is suitable for any clinical situation and is particularly recommended for the *single-stage* technique.



Optimal rigidity

ORANWASH VL, hydrophilic and very fluid

This *very light bodied hydrophilic silicone* is suitable for all *impression techniques*.

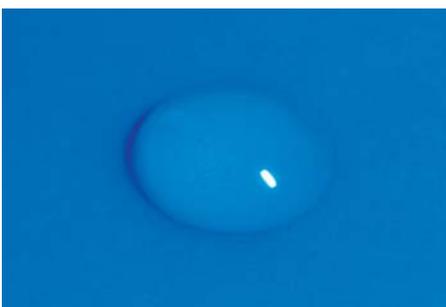

Hydrophilic after 1 minute




Flowability while working

indurent gel and zetaplus system

The ideal colour contrast.

The **INDURENT GEL** catalyst guarantees a *homogeneous mix* for use with the putties and the washes of the **ZETAPLUS SYSTEM**.

Ideal balance between working time and setting time

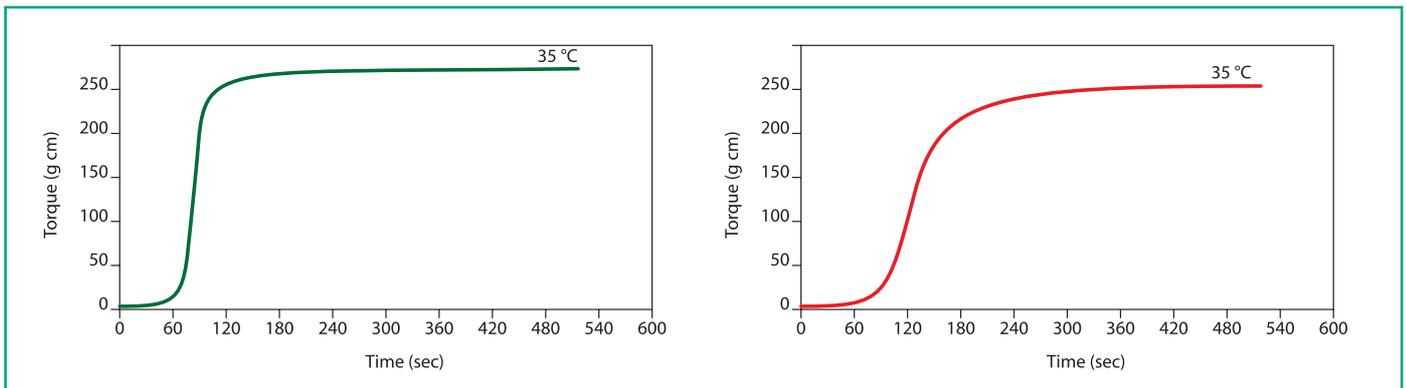

SNAP-SET effect (*zetaplus - zetaplus soft*)

SNAP-SET effect (*oranwash VL – oranwash L*)

TECHNICAL INFORMATION	ZETAPLUS	ZETAPLUS SOFT	ORANWASH VL	ORANWASH L	THIXOFLEX M
Mixing time	30 sec.				
Total working time	1 min. 15 sec.	1 min. 15 sec.	1 min. 30 sec.	1 min. 30 sec.	2 min.
Time in Mouth	3 min. 15 sec.	3 min. 15 sec.	3 min. 30 sec.	3 min. 30 sec.	3 min. 30 sec.
Setting time	4 min. 30 sec.	4 min. 30 sec.	5 min.	5 min.	5 min. 30 sec.
Deformation under pressure (min/max)	2% - 5%	2% - 5%	2% - 8%	2% - 8%	2% - 8%
Elastic recovery	> 98%	> 98%	> 99%	> 99%	> 99%
Dimensional stability	< -0.2%	< -0.2%	< -0.7%	< -0.7%	< -0.7%

Performance **ZETAPLUS SYSTEM**

Clinical

C-Silicone Impression Material



zetaplus system
Reliable in any situation.

SYSTEM**zetaplus**GPLCOMMUNICATION.COM
F121019/A - 10/2009**Packaging**

C100600 ZETAPLUS	1 tub 900 ml (1.53 Kg) + scoop
C100312 ZETAPLUS	1 tub 1800 ml (3 Kg) + scoop
C100468 ZETAPLUS	1 container, 10 Kg + scoop
C100469 ZETAPLUS	1 container, 25 Kg
C100730 ZETAPLUS INTRO KIT L	1 tub Zetaplus 900 ml (1.53 Kg) + scoop + 1 tube Oranwash L 140 ml + 1 tube Indurent gel 60 ml + mixing pad
C100720 ZETAPLUS TRIAL KIT L	1 tub Zetaplus 200 ml + scoop + 1 tube Oranwash L 40 ml + 1 tube Indurent gel 60 ml + mixing pad
C100610 ZETAPLUS SOFT	1 tub 900 ml (1.53 Kg) + scoop
C100612 ZETAPLUS SOFT	1 tub 1800 ml (3 Kg) + scoop
C100613 ZETAPLUS SOFT	1 container, 10 Kg + scoop
C100614 ZETAPLUS SOFT	1 container, 25 Kg
C100740 ZETAPLUS SOFT INTRO KIT VL	1 tub Zetaplus Soft 900 ml (1.53 Kg) + scoop + 1 tube Oranwash VL 140 ml + 1 tube Indurent gel 60 ml + mixing pad
C100710 ZETAPLUS SOFT TRIAL KIT VL	1 tub Zetaplus Soft 200 ml + scoop + 1 tube Oranwash VL 40 ml + 1 tube Indurent gel 60 ml + mixing pad
C100650 ORANWASH VL	1 tube 140 ml
C100660 ORANWASH L	1 tube 140 ml
C100670 THIXOFLEX M	1 tube 140 ml

Catalyst

C100700 INDURENT GEL	1 tube 60 ml
C100390 INDURENT LIQUID	1 bottle 30 ml

Equipment

C303000 ALGHAMIX	Mixing machine for alginate and silicone (230 V)*
C303040 ALGHAMIX	Mixing machine for alginate and silicone (115 V) *

Accessories

U113250	Stainless steel spatula for silicones - sterilizable
D510010	Putty Cut
U112270	Mixing pad, 20 sheets
C303020	Mixing disk for silicones for Alghamix
C303030	Mixing pad for Alghamix, 20 sheets

*Disk for silicone available as an accessory

Branch
Zhermack GmbH Deutschland
 Öhlmühle 10 - D-49448 Marl
 Tel. +49 - 0 54 43 / 20 33 - 0
 Fax +49 - 0 54 43 / 20 33 - 11
 info@zhermack.de
 www.zhermack.com

Branch
Zhermapol Sp. z o.o
 ul. Augustówka 14
 02 - 981 Warszawa - Polska
 Tel. +48 - 22 858 82 72 - 858 73 41
 Fax +48 - 22 642 07 14
 zhermapol@zhermapol.pl
 www.zhermack.com

Branch
Zhermack Inc.
 P.O. Box 4195
 River Edge, New Jersey 07661-4195
 Tel. +1 (877) 819-6206
 +1 (877) 389-8540
 Fax +1 (732) 389 8543
 info@zhermackusa.com
 www.zhermackusa.com

Representative Office
Zhermack UK and Ireland
 21 Balmoral Road
 Melton Mowbray - Leics.
 LE13 1JN
 Tel. +44 - (0)7870 690811
 uk@zhermack.com
 www.zhermack.com

Representative Office
Zhermack Russia
 5/2 Marshala Vasilevskogo St.
 123098 Moscow - Russia
 Tel. +7 916 9301191
 info.russia@zhermack.com
 www.zhermack.com

Zhermack SpA
 Via Bovazecchino, 100 - 45021 Badia Polesine (RO) ITALY
 Tel. +39 - 0425 597611 Fax +39 - 0425 53596 info@zhermack.com www.zhermack.com

Zhermack®
 BEYOND INNOVATION