ENGLISH
<p>Permlastic is a polysulfide impression material for full dentures, partial dentures, inlays, onlays, crowns and bridges.</p> <ul style="list-style-type: none">Type 3: Low consistency - Permlastic Light Bodied Type 2: Medium consistency - Permlastic Regular Type 1: High consistency - Permlastic Heavy Bodied

PRECAUTIONS/CAUTIONS

- Protect patient and staff clothing from contact with both unmixed and mixed components. Contact with clothing will produce staining of clothing.
- Permlastic catalyst contains lead dioxide. Protect against accidental skin contact, eye contact and patient ingestion.
- Kerr Rubber Base Adhesive contains solvent. Use in a well-ventilated area. Avoid contact with skin or eyes. Wear protective gloves and eye protection when using adhesive.
- California Proposition 65 Warning. This product contains a lead compound, a chemical known to the State of California to cause birth defects or other reproductive harm. This product contains a lead compound, a chemical known to the State of California to cause cancer.
- Allow Permlastic to set completely prior to cleaning mixing instruments or placement syringes.
- Avoid cross contamination of base and catalyst. Match cap color to band on neck of tube.

PROPORTIONS

- Permlastic Light Bodied - By volume; equal lengths of base (white) and catalyst (brown) pastes. By weight; 2 grams of base (white) to 3 grams catalyst (brown) paste.
- Permlastic Regular Bodied - By volume; equal lengths of base (white) and catalyst (brown) pastes. By weight; 1 gram of base (white) to 1 gram catalyst (brown) paste.
- Permlastic Heavy Bodied - By volume; equal lengths of base (white) and catalyst (brown) paste. By weight; 3 grams base (white) to 1 gram catalyst (brown) paste.

MIXING TECHNIQUE

Extrude appropriate amounts of base and catalyst pastes onto a mixing pad. Using a spatula, mix materials into each other using a stirring motion. Incorporate thoroughly until a streak free mix is obtained. Mixing should be completed within 45 seconds to 1 minute. Immediately fill tray or syringe.

Viscosity	Light Bodied
Mixing Time	0.75 – 1.0 min
Tray Loading/Syringe Filling	1.0 min
Minimum working time	4 min
Minimum time in the mouth	10 min

Viscosity	Regular Bodied
Mixing Time	0.75 – 1.0 min
Tray Loading/Syringe Filling	1.0 min
Minimum working time	3 min
Minimum time in the mouth	10 min

Viscosity	Heavy Bodied
Mixing Time	0.75 – 1.0 min
Tray Loading/Syringe Filling	1.0 min
Minimum working time	3 min
Minimum time in the mouth	10 min

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ELECTROPLATING

Permlastic impressions may be electroplated.

Silver plating solution	
Silver cyanide	21.43%
Potassium cyanide	57.14%
Potassium carbonate	21.43%

Copper plating solution	
Anhydrous copper sulfate	225-250 gm.
Concentrated H ₂ SO ₄	75 mL
Phenol	10 mL or gm.
Ethanol	25 - 50 mL
Distilled H ₂ O	1.0 liter

GENERAL USAGE GUIDELINES

- Dispense materials in proportions given. Do not vary proportions.
- Do not interchange base and catalyst tube caps.
- Use Kerr Rubber Base Adhesive for all trays (custom or stock)
- Mix until streak free.
- Mix and load tray simultaneously with mixing and syringing of impression area.
- Allow Permlastic to set completely prior to clean-up from instruments and syringes.
- Check set prior to mouth removal.
- Check expiration date prior to use. Do no use expired material.
- Store at ambient temperature.

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Limited Warranty - Limitation of Kerr's Liability

Kerr’s technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr’s product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr’s product. In the event of a defect in material or workmanship, Kerr’s liability is limited, at Kerr’s option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.