



Total Etch leaves its mark

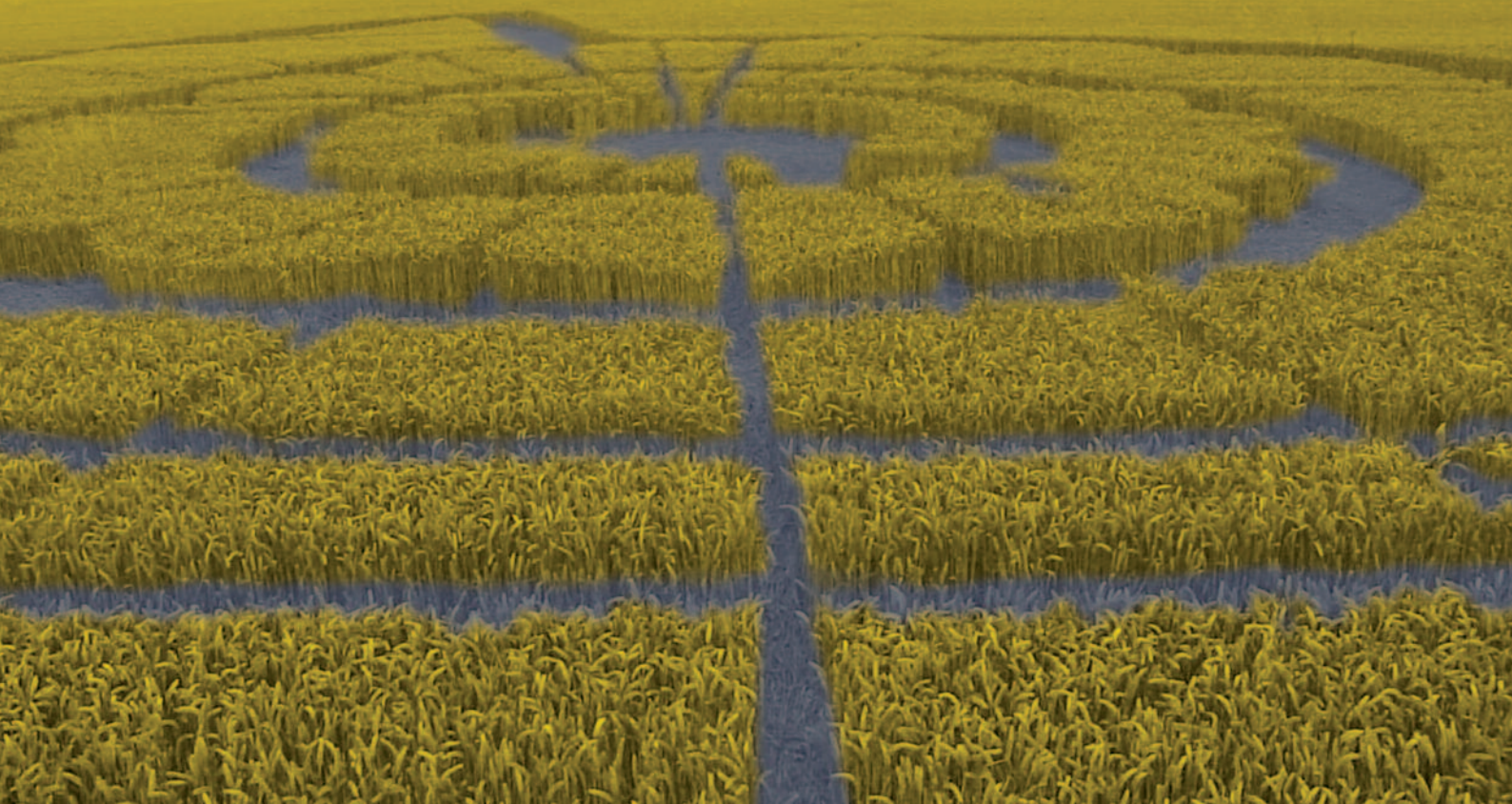


Total Etch

Heightened contrast, easier handling –
the etching gel for a strong bond

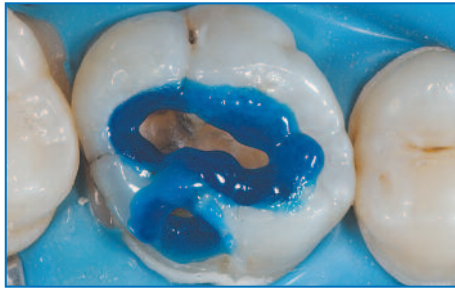

Total Etch is first applied to enamel. The colour of the gel provides a strong contrast to the tooth. Because the gel does not slump, it is easy to apply exactly where it is needed on the enamel without running off into dentin areas.

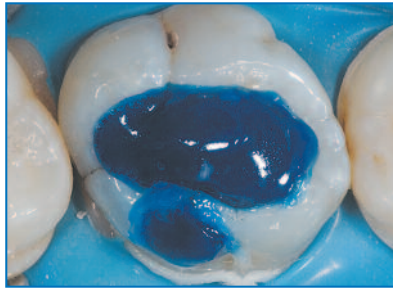

After a brief interval, Total Etch is also applied on dentin.



After it has been etched, rinsed and dried, the enamel has a chalky-white appearance, which aids the operator in checking the etching results.

Consistent viscosity

The viscosity of etching gels may be impaired when they are pressed through the tight nozzle of the dispensing tip. Conventional products often become runny and difficult to apply accurately. The gel flows into the cavity uncontrolled. Due to special filler composition, Total Etch has an ideal and above all consistent thickness that enables it to be applied quickly and effectively.

Distinct blue colour

The blue colour provides a strong contrast to the tooth and allows the gel to be applied accurately. Residue is easy to recognize after rinsing, therefore, minimizing the risk of over-etching.

The advantages of the syringe

- Octagonal finger rest facilitates application and prevents syringe from rolling away
- Angled metal tip for exact application
- Special displacer to prevent material from being wasted

Economical bulk packaging

The convenient Jumbo syringe, which is used to refill syringes, offers a cost-effective solution for keeping adequate amounts of gel in stock. Thanks to the Luer Lock system, the tip can be securely attached and a tight connection to the Jumbo syringe is ensured.

Delivery Forms

Jumbo Assortment Total Etch

1 x 30 g / 2 x 2g

Accessories



Refill Total Etch

2 x 2 g

Accessories



Total Etch Tips

20 tips

