General Information

RelyXTM Luting cement, manufactured by 3M ESPE, is a glass ionomer system comprised of two parts: a powder and a liquid. The powder is a radiopaque, fluoroaluminosilicate glass. The liquid is an aqueous solution of a modified polyalkenoate acid. RelyX Luting cement provides the major benefits of glass ionomer cements - adhesion to tooth structure, fluoride release along with additional benefits of low solubility, improved fracture toughness and a low viscosity, non-stringy, slump-resistant mix.

RelyX Luting cement is a self-cure system using two dark-cure reactions to provide the improved properties.

Indications

RelyX Luting cement is indicated for use when luting:

- Porcelain fused to metal crowns and bridges to tooth structure, amalgam composite or glass ionomer core buildups;
- Metal inlays, onlays or crowns;
- Crowns made with all-alumina or all-zirconia cores such as Procera[®] AllCeram
- Prefabricated and cast post cementation
- Orthodontic appliances.

RelyX Luting cement is not indicated for composite or porcelain inlays or onlays, or composite or all-ceramic crowns not having all-alumina or all-zirconia cores.

Precautionary Information for Patients:
Avoid use of this product in patients with known acrylate^{*} allergies. This product contains substances that may cause an allergic reaction by skin contact in certain individuals. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergic reaction occurs, seek medical attention as needed, remove the product if necessary and discontinue future use of the product.

Precautionary Information for Dental Personnel:

This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured product. If skin contact occurs, wash skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates^{*} may penetrate commonly used gloves. If product contacts glove, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

Contains Potassium Persulfate. May produce an allergic reaction.

Contains Potassium Persulfate that may trigger an allergic respiratory reaction in certain individuals. This product may not be appropriate for use in those individuals with known sensitivity to sulfites, since a cross-reaction may occur with Potassium Persulfate.

Instructions for Use

1. Remove the provisional restoration and all residual temporary cement. Thoroughly clean the preparation with an oil-free pumice paste. Rinse with water and let dry.

Note: Do not desiccate the tooth. Desiccation of tooth structure is believed to cause post operative sensitivity in some individuals.

2. Pulp protection: Use a hard setting calcium hydroxide material with near exposures.

NOTE: Use of some tooth preparation treatments is contraindicated as their residue may adversely affect bond strength or the setting reactions (e.g., Tubulicid Red).

Contact your local 3M ESPE representative or the 3M ESPE Customer Hotline for specific details.

3. Casting preparation: Thoroughly clean interior surfaces of cast crown, inlay or onlay.

4. Dispense powder and liquid: The standard powder/liquid ratio of 1.6:1 by weight can be obtained by using an equal number of level scoops of powder and liquid drops.

Three scoops of powder and three drops of liquid will provide an adequate amount of material to seat one typical crown.

Shake the jar to fluff the RelyX Luting powder before dispensing. Insert the scoop into the jar, overfill it with loosely packed powder and withdraw it against the plastic lever to remove excess powder and obtain a level scoop. Dispense the desired number of powder scoops onto the mixing pad.

Note: The glass ionomer powders are sensitive to high humidity. Store with jar caps securely tightened and away from high humidity.

To obtain a proper liquid drop size, hold the RelyX Luting cement liquid vial vertically with the dropper tip down and without the tip contacting the mixing pad. Squeeze the vial to dispense the desired number of liquid drops onto the mixing pad.

5. Mixing: Using a cement spatula, mix the powder into the liquid. To minimize water evaporation and maximize working time, confine spatulation of the powder and liquid to a small area of the mixing pad. All of the powder should be incorporated into the liquid within 30 seconds.

6. Working time of the standard powder/liquid ratio is at least 2.5 minutes from the start of mix at a room temperature of 73°F (23°C). Higher temperatures and vigorous spatulation will shorten working time. Lower temperatures will lengthen working time.

Note: Working time can be lengthened by using refrigerated liquid or by mixing on a cold slab.

7. Crown placement: Load the crown by spreading a layer of the cement over all the interior surfaces of the crown. Seat crown. Maintain pressure on crown to maintain position during setting process.

8. Clean up excess. Excess material can be removed when cement reaches a waxy stage after a minimum of 3 minutes from placement in the mouth (37°C or 98°F). Use a suitable instrument for this process.

Cement will set hard within 10 minutes after placement in the mouth.

Storage and Use

1. Shelf life at room temperature is 24 months. See outer package for expiry date.

2. The glass ionomer system is designed to be used at room temperatures of approximately 70-75°F (21-24°C). Replace caps immediately after use.

3. Glass ionomer powders are sensitive to high humidity. Store with jar caps securely tightened and away from high humidity.

4. Do not substitute VitremerTM Core Buildup/Restorative powder or liquid, manufactured by 3M ESPE, for RelyX Luting cement powder or liquid.

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty
3M ESPE warrants this product will be free from defects in material and manufacture. 3M ESPE MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M ESPE's sole obligation shall be repair or replacement of the 3M ESPE product.

Limitation of Liability
Except where prohibited by law, 3M ESPE will not be liable for any loss or damage arising from the product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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Indications

RelyX Luting cement is indicated for use when luting:

- Porcelain fused to metal crowns and bridges to tooth structure, amalgam composite or glass ionomer core buildups;
- Metal inlays, onlays or crowns;
- Crowns made with all-alumina or all-zirconia cores such as Procera[®] AllCeram
- Prefabricated and cast post cementation
- Orthodontic appliances.

RelyX Luting cement is not indicated for composite or porcelain inlays or onlays, or composite or all-ceramic crowns not having all-alumina or all-zirconia cores.

Precautionary Information for Patients:

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Procera[®] is a registered trademark of Nobel Biocare AB.